THERALINE

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WHY DOES THE BACK OF MY BABY'S HEAD BECOME MISSHAPEN?

With good reason, midwives and doctors recommend you lay your baby on his/her back to sleep. Thanks to this recommendation the cot death rate in all western European countries has dropped noticeably. Unfortunately, though, the bones of the skull are so soft during the first months of life, that with long periods lying down there is often a clearly visible flattening and/or misshaping of the head (flat head syndrome or Positional Plagiocephaly). Often a bald patch also appears on the back of the head.

PROTECT YOUR BABY AT THE RIGHT TIME

Anyone who looks carefully can see that nowadays many babies' heads become, to a greater or lesser extent, misshapen or deformed. In normal cases the deformities grow off in the baby's first year due to the "growth pressure" of the brain. Yet often the deformities are simply accepted for the better protection of the baby, and seen as merely "the lesser of two evils".

Luckily the Theraline Baby Pillow (size 1 prevention & safety) now offers your baby both comfort and highest safety during sleep and, combined with this, it helps to preserve the natural head shape. The Theraline Baby Pillow helps to avoid deformities brought about by sleeping position equally as well as a cosy feather pillow would do, but without the associated risks. If head deformations have already formed, the Theraline baby pillow size 2 supports the treatment and regression.

HOW DOES THE THERALINE BABY PILLOW WORK?

A simple idea and a unique design ensure safety and comfort for your baby when sleeping and lying. Anatomically shaped and perforated high-tech fabric supports your baby's little head evenly and effectively. Used right from the beginning a deformity of the head can mostly be prevented. For babies already affected by deformities the Theraline Baby Pillow can help with the faster correction of the deformity*. And thanks to the special fabric, it does so safely for your baby.

With a special arrangement of different types of fabric, the Theraline Baby Pillow is so open that, even when your baby begins to turn and the head comes to rest in the pillow's hollow, he or she can still breathe easily through. Of course your baby can still move his/her head completely freely despite the pre-shaped hollow, so that development, the urge to move and natural curiosity are not stifled.

IF YOUR BABY IS VERY RESTLESS WHEN SLEEPING

During the first months of life, the Theraline baby pillow usually remains in position right under the baby's head, which means the baby will not suffer from any head deformities, and most birth-related or pre-natal deformities will regress very quickly*.

Once babies reach 2-5 months of age, or if they suffer from existing deformities that have resulted from their sleeping position, they are usually very restless when sleeping, meaning the baby pillow may shift. Therefore, the baby's head must be placed back on the pillow and repositioned frequently to aid a fast regression of the deformities. An extra effort your baby will thank you for.

Safety first: securing the Theraline baby pillow, with a hush cloth or under a fit-

ted sheet for example, will prevent it from slipping, however, it also reduces the breathability of the special fabric. This is a serious risk once your baby starts turning without assistance.

TIPS FOR DEALING WITH HEAD DEFORMITIES*

- The American Academy of Pediatrics recommends „Back to Sleep, Tummy to Play“.
- If your baby tends to look in the most typical direction when in bed, i.e. towards the door or the window, then turn your Baby more often, so that the other side of the head takes the weight.
- Once your baby has fallen asleep, lay his/her head down again onto the non-deformed side.
- Use the Theraline Baby Pillow whenever possible to achieve an optimum distribution of pressure on the baby's head.

CHARACTERISTICS OF THE 3D SPACER FABRIC

- Extremely high air circulation in all directions** – even when weighed down
- Pleasant feeling to the padding
- Optimum elasticity
- Temperature equalizing
- Smooth and skin-friendly surface

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Attention:

Avoid pressure onto the larynx due to the tilt position of the baby's head.

After placing the baby's head onto the Theraline baby pillow make sure that head and body form an approximately straight line. Some head shapes (especially widely shaped heads) do not sink into the depression deep enough so that the baby's head may be positioned 1,5 -2,5 cm too high. A subsequent enlargement of the depression can be achieved. Please contact us for further information.

PERIOD OF APPLICATION

Size 1 Prevention & Safety*: up to 7 kg

Size 2 Regeneration & Safety*: from 7 kg onwards

Attention: Wash at 60 °C before the first use!

* Please always consult your pediatrician if you observe a flat head syndrome at your baby. No claim is made to the variety of individual cases and conditions which may cause a flat head syndrome.

Note: The Theraline baby pillow serves for prophylactic use (size 1) and regeneration (size 2). They can only accompany a therapy against head deformation. Especially with indication of brachycephaly always consult a pediatrician!

** To ensure absolute safety please make sure to abstain from any pillow covers, towels or any kind of cloths.

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